



QR srl con socio unico

Via Silvestrini, 20

37135 Verona, I



T +39 045 8202727

+39 045 583500

F +39 045 8203040

Info@qrverona.it

www.qrverona.it

NewTom 3G

Dedicated CB3D protocol

Patient Scan: select 'Patient Scan' in Scan Manager

Radiographic guide: select 'Denture Scan' in Scan Manager

Study reconstruction

Primary Reconstruction: "High High Resolution" (or "High Resolution")

Study Reconstruction: - 0.3 or 0.4mm axial thickness

- No inclination: 0°

CB3D converter settings (For recent Updates in Procera Software, please, contact your Nobel reference)

Threshold bone: 400 - 650; Smoothing: radius: 3, 3, 3; standard deviation: 12, 12, 12

Threshold radiographic guide: ± 0

Threshold markers in patient scan: 1950 – 2350

Threshold markers in radiographic guide scan: 1950 – 2350

NewTom VG/VGi (Hi Res Zoom)

Dedicated CT protocol

Patient Scan: select 'Patient Scan' in Scan Manager

Radiographic guide: select 'Denture Scan' in Scan Manager

Study reconstruction

Primary Reconstruction: "Standard"

Study Reconstruction: - 0.3 or 0.4mm axial thickness

- No inclination: 0°

CT converter settings (For recent Updates in Procera Software, please, contact your Nobel reference)

Threshold bone: 400 - 650; Smoothing: radius: 3, 3, 3; standard deviation: 12, 12, 12

Threshold radiographic guide: ± 0

Threshold markers in patient scan: 1950 – 2350

Threshold markers in radiographic guide scan: 1950 – 2350

NewTom